 **WONDER**

Axan

RADICAL MUSCLE TONING

Dr. Foo Wing Jian
LCP Certified Aesthetic Physician
Premier Clinic
Dip. Aesth (AAAM), Cert. Derm (MAL)

Made in Spain
Made for You

WONDER

Axan

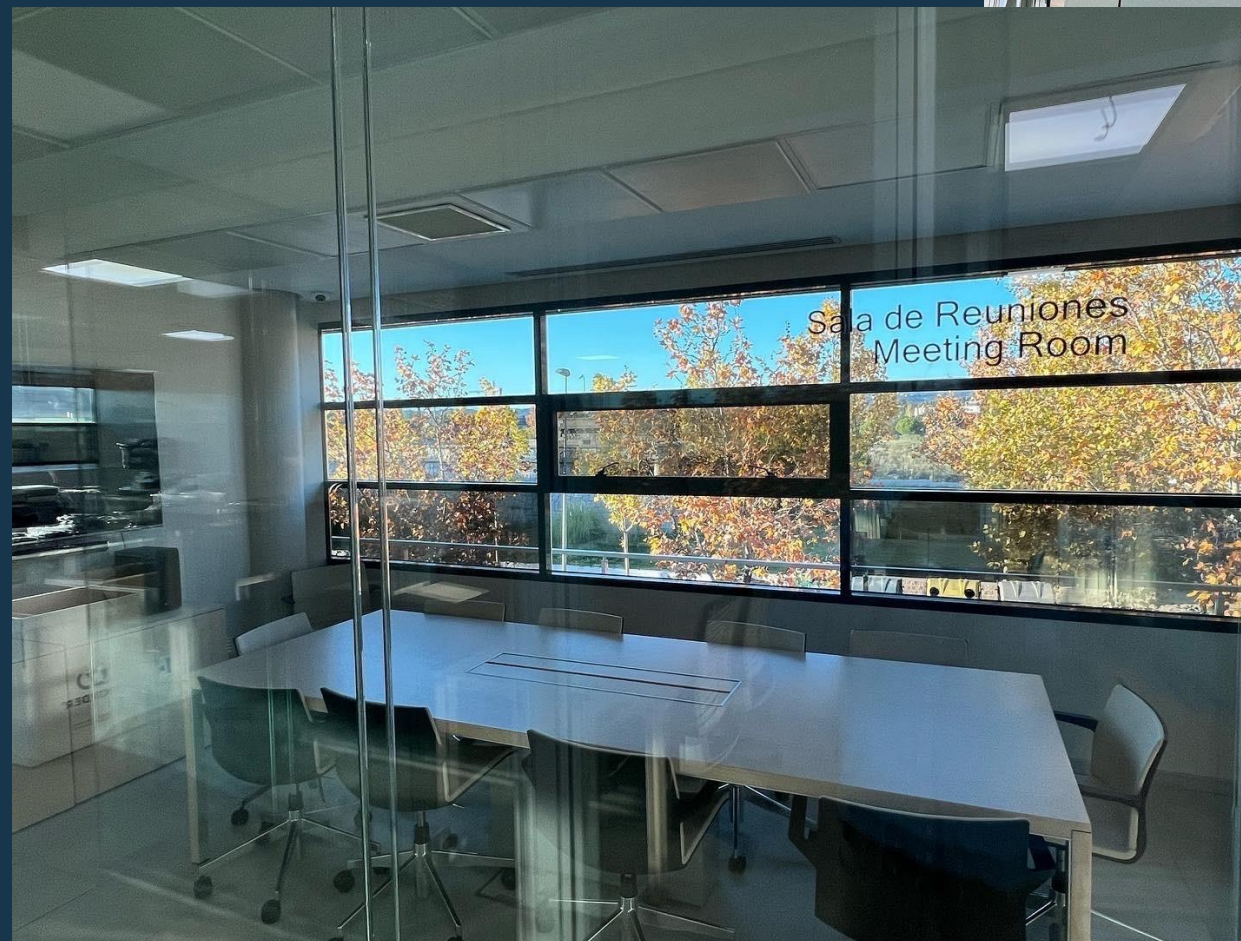
WONDER IS THE N°1
SPANISH TECHNOLOGY FOR
MUSCLE BUILDING AND FAT
BURNING ALREADY PRESENT
IN PRESTIGIOUS CLINICS
IN 60 COUNTRIES.

**SOMETHING IMPORTANT
UNITES THE WONDER
TEAM:**

**THE LOVE OF CHALLENGES
AND THE DESIRE TO
CHANGE PEOPLE'S LIVES.**



**Made in Spain
Made for You**

THE STORY OF WONDER

2019

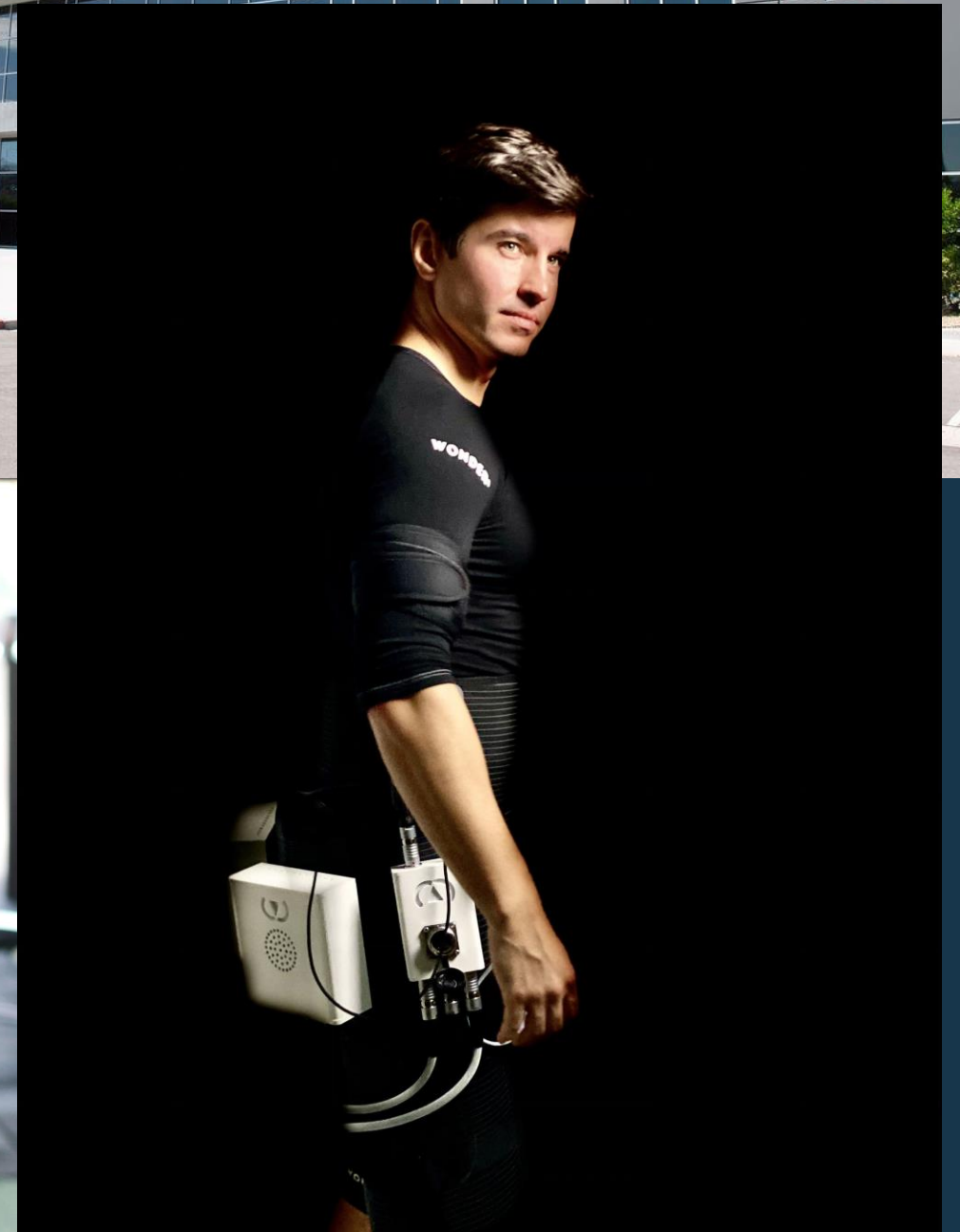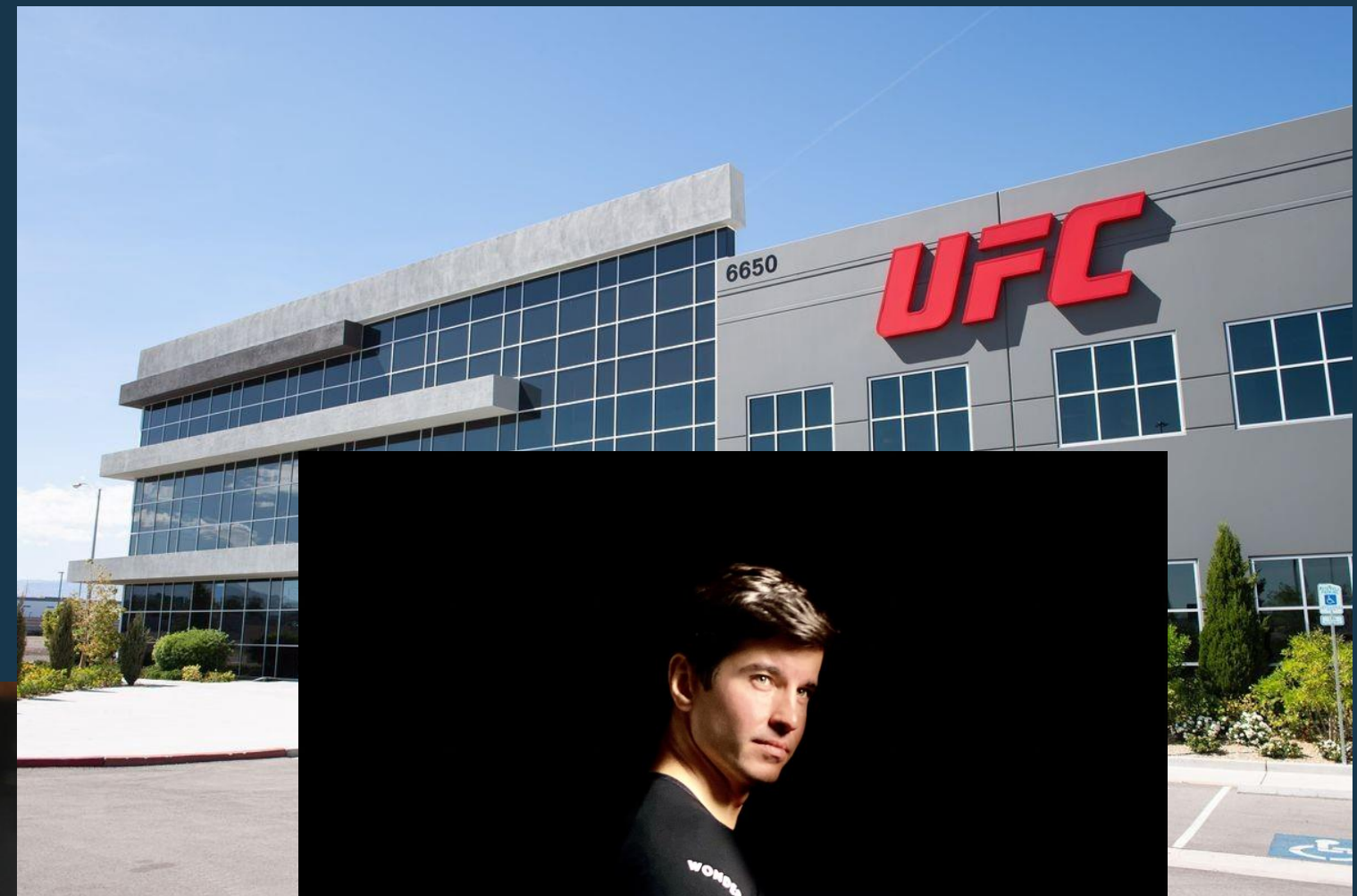CREATION
IN THE UFC
HEADQUARTERS
IN NEW YORK

Made in Spain
Made for You

2020
THE YEAR
OF COVID.
150 DEVICES
SOLD TO HELP
RECOVERY
AFTER ILLNESS

Made in Spain
Made for You

WONDER
Axon

2021

**POST - COVID WORLD.
RE-OPENING CLINICS
AND SPAS.**

**600 DEVICES
DELIVERED IN 35
COUNTRIES**

**Made in Spain
Made for You**

2022-2023

WONDER IS WORLDWIDE KNOWN.

PARTICIPATION IN THE MOST IMPORTANT EXHIBITIONS: IMCAS, AMWC, COSMOPROF, ETC.

MORE THAN 1000 DEVICES SOLD IN 60 COUNTRIES

Made in Spain
Made for You

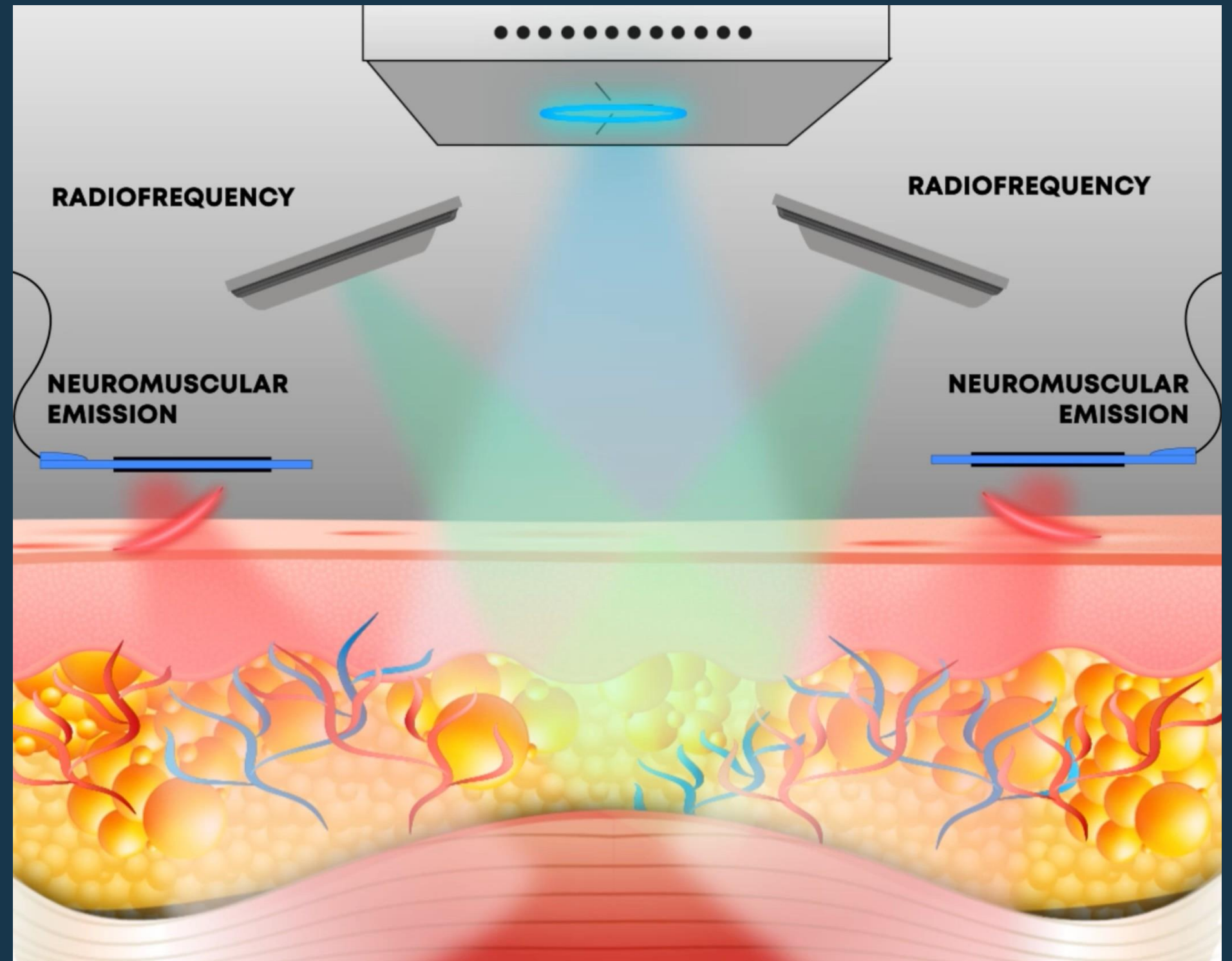
TECHNOLOGY


TRIPLE ACTION TRIPLE EFICACITY

**ELECTROMAGNETIC
+ NEUROMUSCULAR
+ RADIOFREQUENCY**


TRIPLE ACTION. TRIPLE EFFICACY.

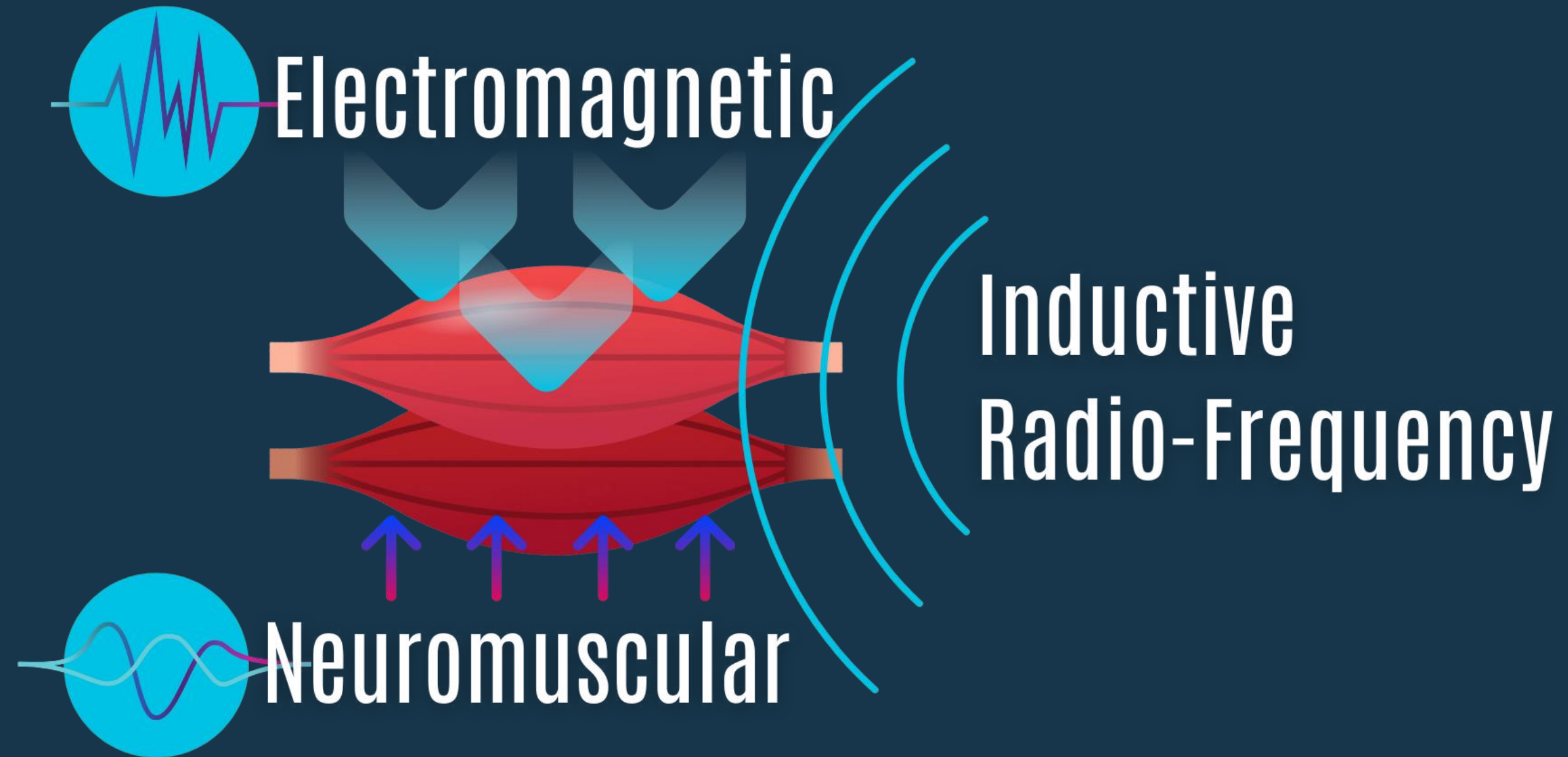Axon

NEUROMUSCULAR

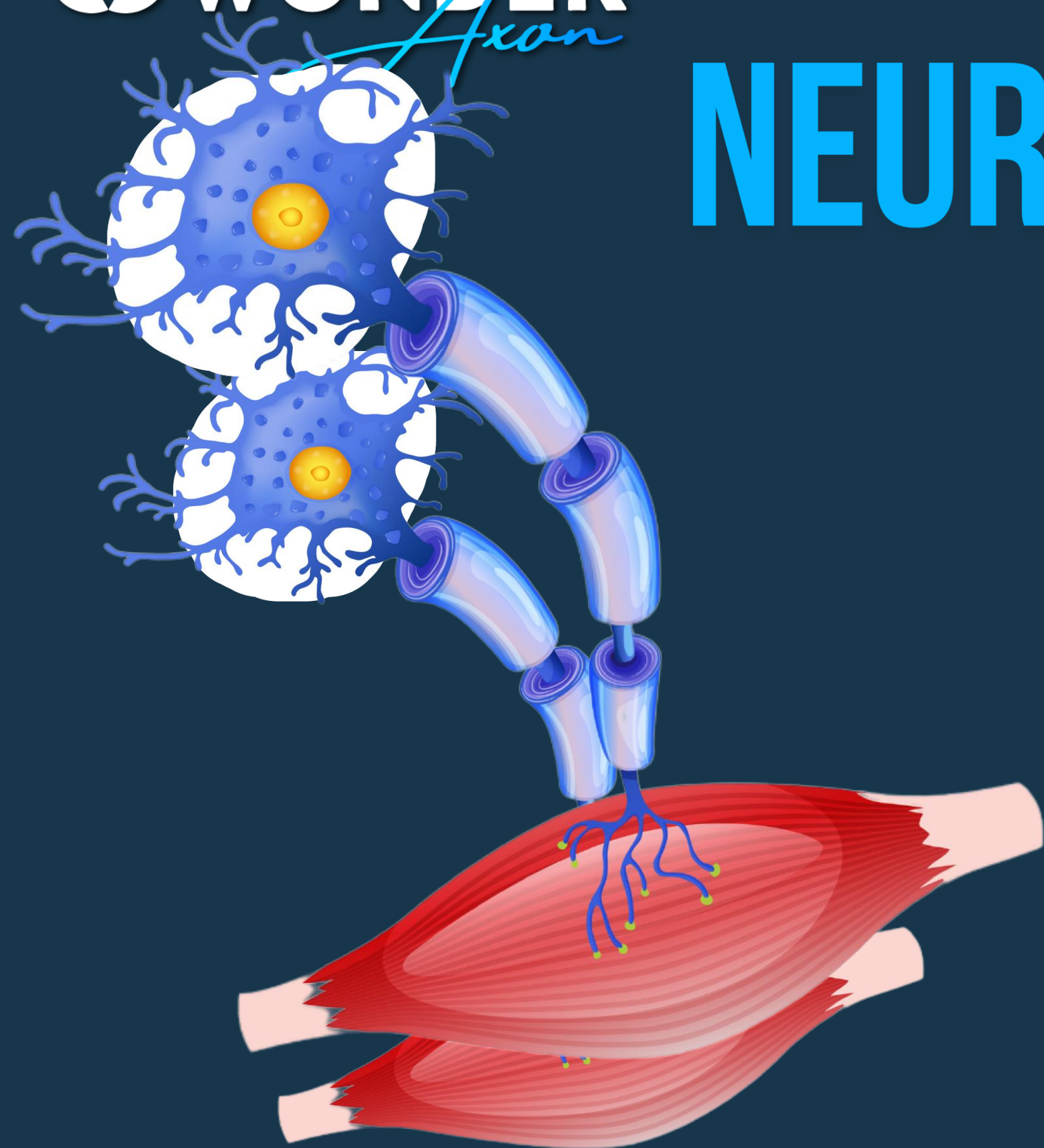These waves are focused exclusively on the motor neuron, produces an hyperexcitation and triggers up to 60,000 contractions.

NEUROMUSCULAR

IT'S THE MOST IMPORTANT EMISSION IN WONDER DEVICES. IT IMITATES THE SIGNAL FROM THE BRAIN AND CONCENTRATE THE EMISSION SPECIFICALLY IN THE MOTONEURONE. WONDER IS NOT AN ELECTROSTIMULATION DEVICE.

WONDER EMITS NEUROMUSCULAR WAVES. THESE WAVES ARE FOCUSED EXCLUSIVELY ON THE MOTOR NEURON, PRODUCES AN HYPEREXCITATION AND TRIGGERS UP TO 52,000 SUPRAMAXIMAL CONTRACTIONS.

RADIO FREQUENCY

1 REGENERATIVE. IMPROVES THE SKIN (TEXTURE, SMOOTHNESS, STRETCHING EFFECT).

2 LIPOLYPTIC. INCREASES THE TEMPERATURE IN DEPTH AND HELPS TO ELIMINATE FAT.

3 SEDATIVE. BY HEATING THE NERVE ENDINGS, THE MUSCLES RELAX AND WE CAN USE MORE POWER AND WORK DEEPER WITH VERY PLEASANT SENSATIONS.

ELECTROMAGNETIC

ELECTROMAGNETIC EMISSION PRODUCES VIBRATION OF MUSCLE FIBERS.

IT'S USED IN WONDER TO PROMOTE THE DEVELOPMENT OF MUSCLE TONE AND BONE DENSITY.

WONDER IS NOT AN HIFEM DEVICE. ITS A PLATFORM THAT COMBINES ALL OF THREE MORE EFFECTIVE AND POWERFUL EMISSIONS.

RECORD

PULSE WIDTH

1000 MICROSECONDS
= 20 CMS

MAXIMUM

PENETRATION

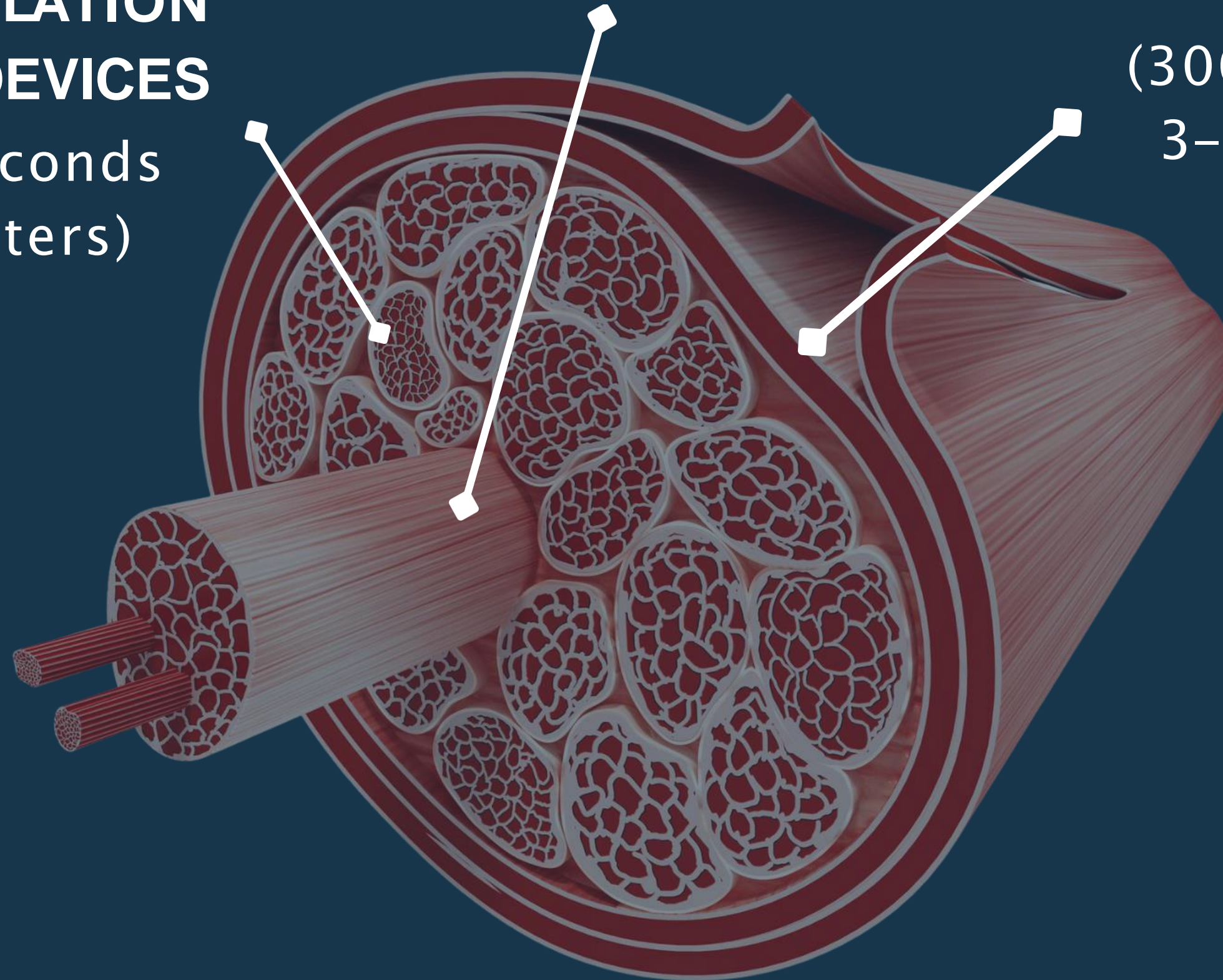
**ELECTROSTIMULATION
DEVICES**

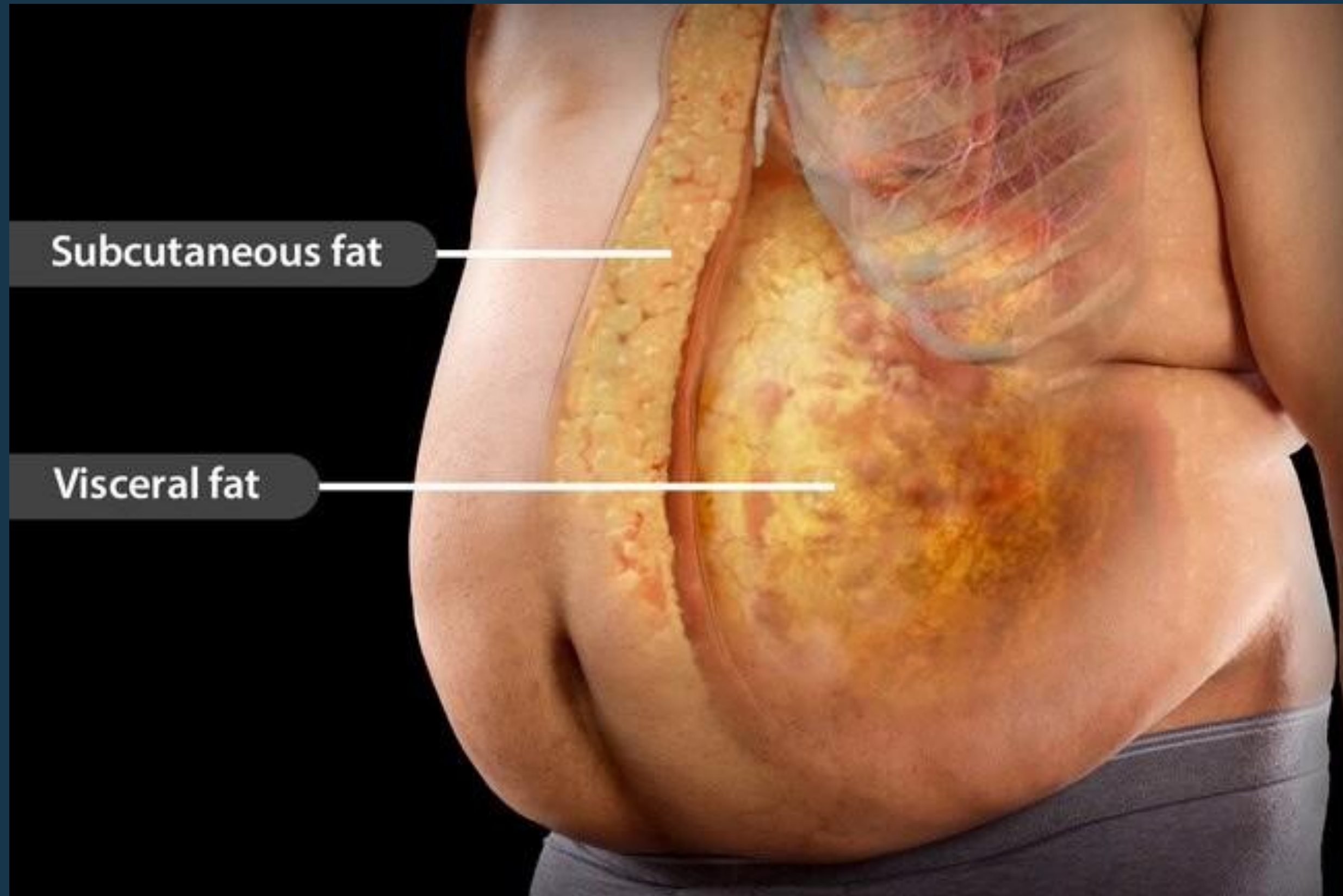
(350 microseconds
5–6 centimeters)

WONDER
(1000 microseconds
18–20 centimeters)

ELECTROMAGNETISM DEVICES

(300 microseconds
3–5 centimeters)

STATISTICS


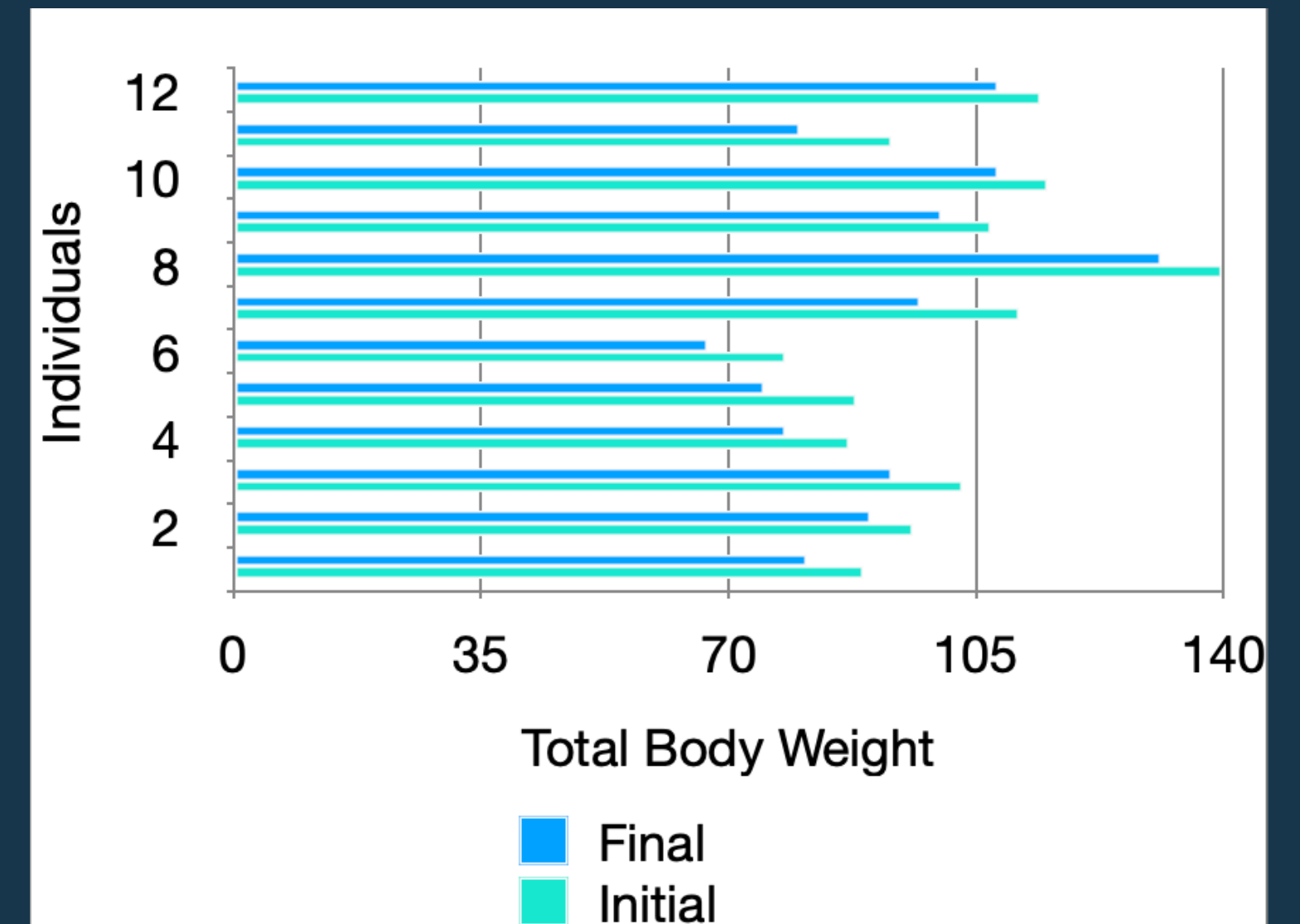
María A. Barajas, MD. Santander University Hospital, Colombia.

REDUCTION OF TOTAL WEIGHT.

All the subjects had a weight loss corresponding mainly to subcutaneous fat, with a minimum weight loss of 5.2 kg and a maximum loss of 14.7 kg, with a average of 9.5 kg in the group. There was a loss of 5- 10% of the initial weight in 60% of the participants, and a loss of 10- 15% of the initial weight in the remaining 40%. Furthermore, weight loss was found to be slightly higher in the female participants.

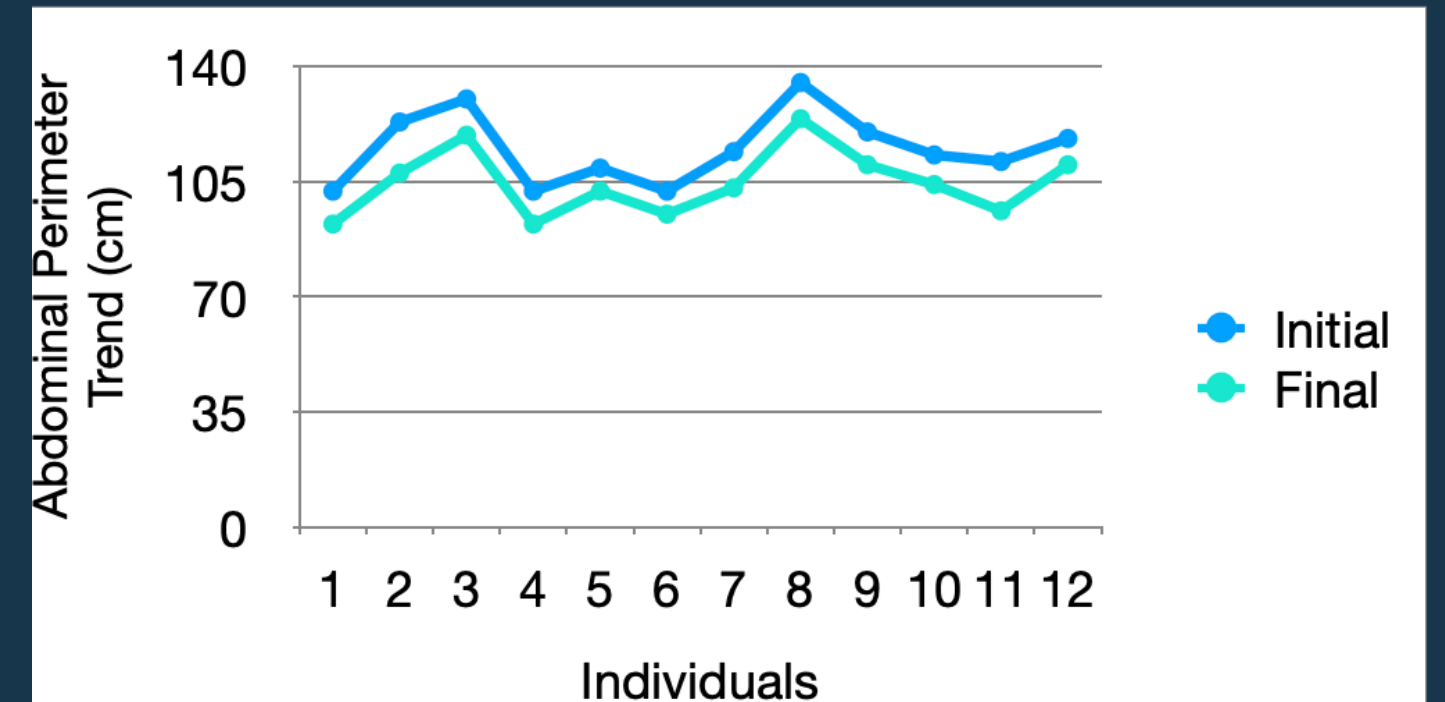Loss of predominantly fatty tissue at the minimum abdominal level was 7 cm with a maximum of 15.5 cm, for a mean of 10.3 cm, without significant changes between both sexes. The trend of the reduction of the abdominal perimeter is observed, which was symmetrical and proportional among all the individuals.


WAIST CIRCUMFERENCE MEASUREMENT.

When comparing the average results between the first and tenth Wonder sessions for waist diameter, the percentage difference was 5.35 centimeters, resulting in significant visible changes. The patients who experienced less waist reduction did so by 3 centimeters and we found patients with a waist reduction of more than 10 centimeters. This reduction is comparatively very significant.

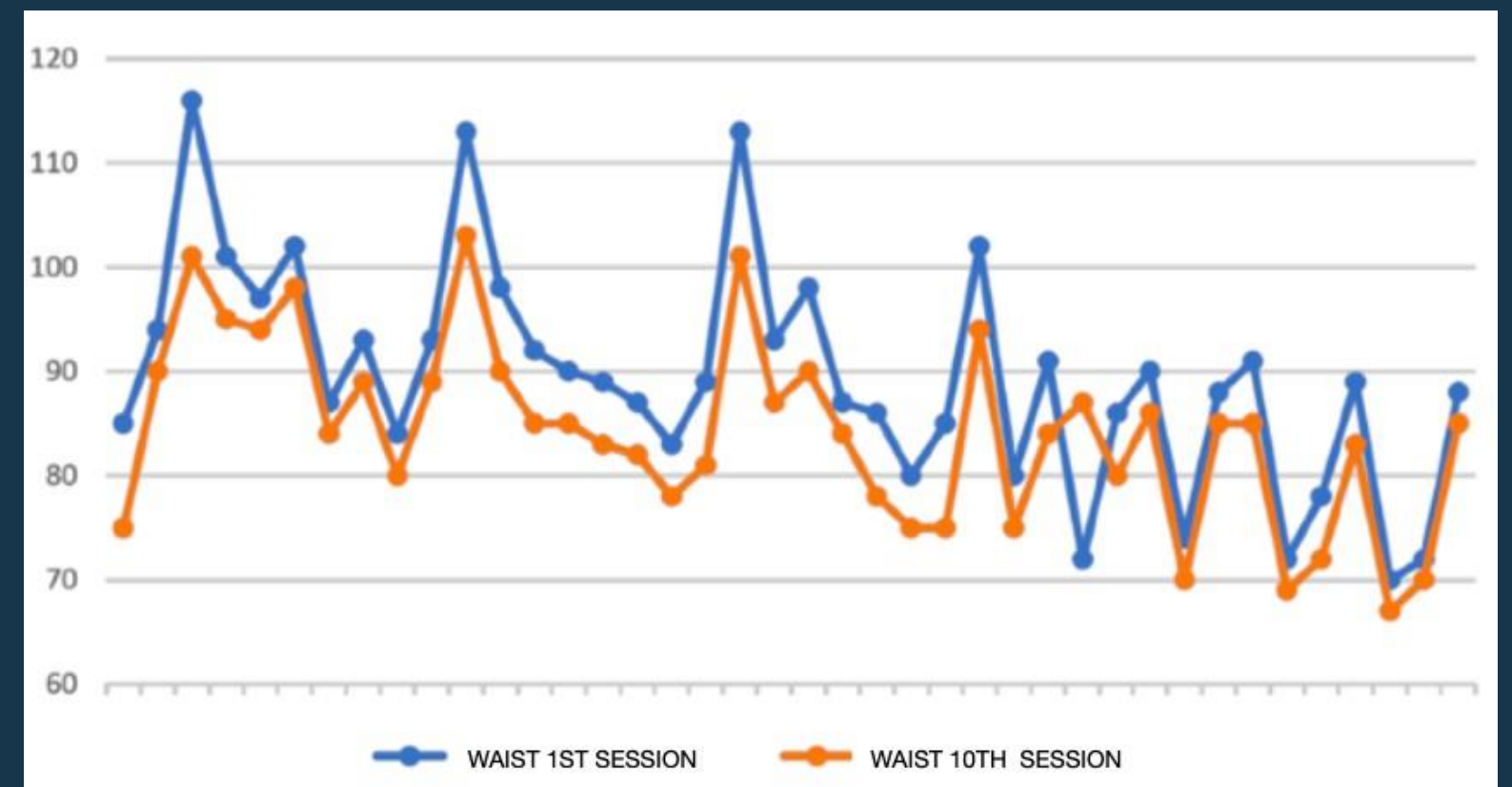Dr. Odalys González . Aesthetic Surgeon.
Master in Aesthetic Medicine.
Guadalajara, México


ABDOMEN CIRCUMFERENCE.

When comparing the average results between the first and tenth session of Wonder for the diameter of the abdomen, the percentage difference was 11.51 centimeters, resulting in significant visible changes. The patients who had less abdomen reduction, was 2 centimeters and we found patients with abdomen reduction of more than 10 centimeters.

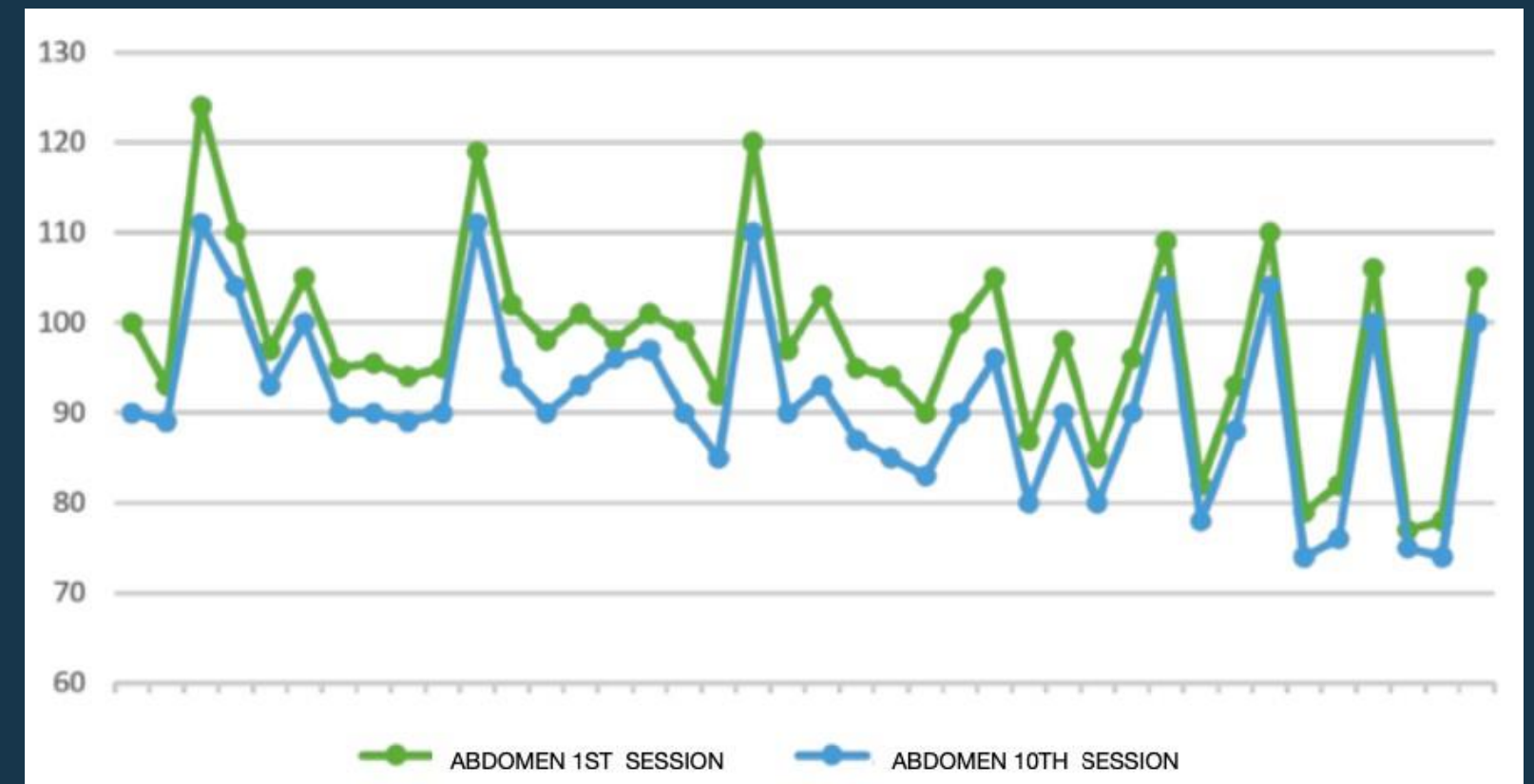Dr. Odalys González . Aesthetic Surgeon.
Master in Aesthetic Medicine.
Guadalajara, México

THE WONDERWEAR

WHY DO WE USE WONDERWEAR?

1 HYGIENIC

2 PRIVACY

3 EXCLUSIVITY

WONDER
Axon

**ENDORSED BY
SPECIALISTS**

Hundreds of prestigious professionals have chosen Wonder to respond to the growing demand for body toning treatments.

FEASIBILITY STUDY FOR THE USE OF ELECTROMAGNETIC AND NEUROMUSCULAR MUSCLE STIMULATION AS A COMBINATION THERAPY FOR BODY CONTOURING, WITH WONDER DEVICES.

María A. Barajas, MD. Santander University Hospital, Colombia.

Copyright: ©2022. The use, distribution and reproduction without express authorization is not allowed.

BACKGROUND: Electromagnetic and Neuromuscular muscle stimulation are treatments currently used separately by different disciplines such as neurology, physiotherapy and sports medicine.

OBJECTIVE: To establish the efficacy and safety of the joint use of electrical and electromagnetic muscle neurostimulation for noninvasive contouring of the abdomen, hips, and legs.

METHODOLOGY: This was a non-randomized, prospective, open and multi-centered feasibility study, in which 12 patients from Madrid, Spain were chosen to undergo a working protocol of 16 sessions of the Wonder Prestige device in a period of 8 weeks and its effect on three body parts- abdomen, buttocks and thighs.

Feasibility study for the use of electromagnetic and neuromuscular muscle stimulation

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Original informational scientific article

General information on the application of technology Wonder® Strengthening Muscle Therapy.

Omar Antonio Gonzalez Escobar.*
Bachelor of physiotherapy and integral rehabilitation.

I. Introduction.

The human body is an electromagnetic machine whose main source of energy comes from terrestrial magnetism, although this is not the only way. Oxygen, food, cellular activity, physical and mental activity, fluid and biochemical factors represent the constant production of biomagnetic energy. Therefore, the treatment of many diseases and the emergence of treatment methods aimed at improving the physical

This concept served to explain the properties of supermagnetic materials such as magnetic stone. Years of research allowed Dr. Kioichi Nakagawa, a Japanese scientist, to formulate his theory of "magnetic field deficiency syndrome". This syndrome occurs in people who spend most of their time inside buildings, the support of which consists of huge gratings. These gratings do not allow the lines of force of the Earth's

General information on the application of Wonder® Strengthening Muscle Therapy

Omar Antonio González Escobar M.D.
Bachelor of physiotherapy and integral rehabilitation.

Original contribution.

Physiological Approach to Muscle and Toning Therapy with Wonder® MT Device.

Amparo Cruz Onos MD
First Degree Specialist in General Medicine.
Second Degree Physiology Specialist. Lucero Clinic, Madrid.

Copyright: © 2020. Use, distribution and reproduction are permitted provided the author and source of the article are indicated.

I. INTRODUCTION

Medical and cosmetic body remodeling treatments based on increasing muscle mass in specific areas of the body such as the buttocks, legs and arms are experiencing an exponential increase in demand from patients of all ages, genders and morphologies.

Physiological Approach to Muscle and Toning Therapy with Wonder® MT Device

Amparo Cruz Onos M.D.
First Degree Medical Specialist in Comprehensive General Medicine.

Original informative article

CLINICAL STUDY OF PELVIC FLOOR MUSCLE INCREASING PROCEDURE USING WONDER® MT.

Dr. Cesar Caballero Perez
Internal Medicine Specialist
College number 26825
Teaching Hospital "Dr. Joaquin Castillo Duani"

I. Introduction.

The pelvic floor is a system of muscles and ligaments that close the floor of the abdomen, keeping the bladder, uterus, and rectum suspended against gravity.

The fibers of the pelvic floor are directed downward and outward, forming a "hammock" that collects the insides of the pelvis. Muscles intersect and surround 3 openings: urethra, vagina and anus, helping to control the sphincter.

Weakening of the pelvic floor causes one or

Perineal rehabilitation is challenging because, unlike any other muscle in the body, the contraction of the perineal muscle is not visible to the eye, as it is an internal muscle that is more difficult to monitor than other muscles. However, it is very important for the health of women and their quality of life.

Among the most common causes of pelvic floor weakness are:

- Pregnancy

Clinical study of pelvic floor muscle increasing procedure using Wonder® MT.

Cesar Caballero Perez M.D.
Internal Medicine Specialist

Original informational scientific article

Muscle Mass Increase And Fat Loss Therapy With Wonder Technology: Electromagnetic and Electrostimulation Combined Emissions.

Obdulia Ramirez Milan M.D.
First Degree Medical Specialist in Comprehensive General Medicine.
II Degree Specialist in Clinical Embryology. Consulting Professor.

I. Introduction.

Non-invasive body image enhancement, slimming and toning treatments are increasingly in demand and popular. Statistics show an increase of more than double between 2012 and 2016.

Therapies such as cryolipolysis, radiofrequency, low-level laser and HIFU (High Intensity

of treating various medical indications, such as neurology, psychiatry, and physical therapy. Thanks to its non-thermal and non-ionizing nature, its application is considered safe. However, it is not as widely used or effective in the muscle building process as electrical stimulation is.

Muscle Mass Increase And Fat Loss Therapy With Wonder Technology:

Obdulia Ramírez Milan M.D

First Degree Medical Specialist in Comprehensive General Medicine.

Effect of WONDER Muscle Recovery Therapy among Middle-Aged People: Analytical Study on 40 patients.

Author:
Dr. Odalys González Álvarez.
Aesthetic Surgeon. Master in
Aesthetic Medicine.

Coauthor:
Dr. Madelen Guillen Vargas.
First Degree Specialist in
Anesthesiology. Master in
Medical Emergencies and
Pharmacoepidemiology.

Guadalajara México

Abstract: The device based on electromagnetic emissions (HIFEM) and electrical emissions, whether considered neurostimulation (NMS), has proven to be effective and safe for shaping the body. The authors investigated the efficacy of a new device that synchronously delivers both emissions to build muscle and reduce fat in various parts of the body simultaneously.

Method: A retrospective analytical study was carried out in 40 patients, who received treatment with the Wonder equipment. The mean age was 44.4 years, with a predominance of males. The patients underwent 10 treatment sessions, taking

Effect of WONDER Muscle Recovery Therapy among Middle-Aged People

Odalys González Álvarez M.D.

Aesthetic Surgeon. Master in Aesthetic Medicine.

Madelen Guillén Vargas M.D.

Master in Medical Emergencies and Pharmacoepidemiology.

Original informational scientific article

The importance of buttock muscles and the Wonder® strengthening muscle tonic therapy.

Dr. Yixania Valdivia Calero MD
First-degree medical specialist in general complex medicine.

I. Background.

A muscle is an organ that consists mainly of muscle tissue, as well as connective tissue formed by contractile spindle-shaped cells that shorten and lengthen in response to a nerve stimulus.

Muscles have the property of contraction, a process in which muscle proteins, calcium and ATP are involved; during this process one of the mechanisms of

The term «buttock muscle» is used as a euphemism, although in reality it refers to a set of muscles located in the lower part of the spine, which consists of the large buttocks muscle, average buttocks muscle and small buttocks muscle.

Anatomically, the buttocks are deposits of fat applied to the muscles of the buttocks (large buttocks muscle, average buttocks muscle and small buttocks muscle). Buttocks are separated by the so-called interdental fissure, in which the anus is located. When sitting, support comes from the sciatic

The importance of buttock muscles and the Wonder® strengthening muscle tonic therapy

Yixania Valdivia Calero M.D.

First-degree medical specialist in general complex medicine.

ADVANTAGES

THE

WONDERSUIT

12

NEUROSTIMULATION
SENSORS

2

ELECTROMAGNETIC
PANELS

2

RADIOFREQUENCY
EMISSORS

THE SOFTWARE

CONTROL OF SESSIONS
REAL-TIME MANAGEMENT

CONNECTION TO WIFI
REMOTE CONTROLLED

AUTOMATIC PROGRAMS
SELF-REGULATION

**WIRELESS.
EASY.
PREDICTIVE.**

**All control of Wonder is
in your hands.
From distance.
Your presence is no
longer essential.**

IS A MATTER OF SENSATIONS

We paid special attention to offering our users an unforgettable **UNIQUE EXPERIENCE**.

Treatment with of Wonder is exciting and pleasurable. Customers always ask for more!

TOTALLY SAFE

Without risks or complications.
The powerful treatment
with Wonder produces only
muscle pain the days after
treatment and requires no
recovery time.

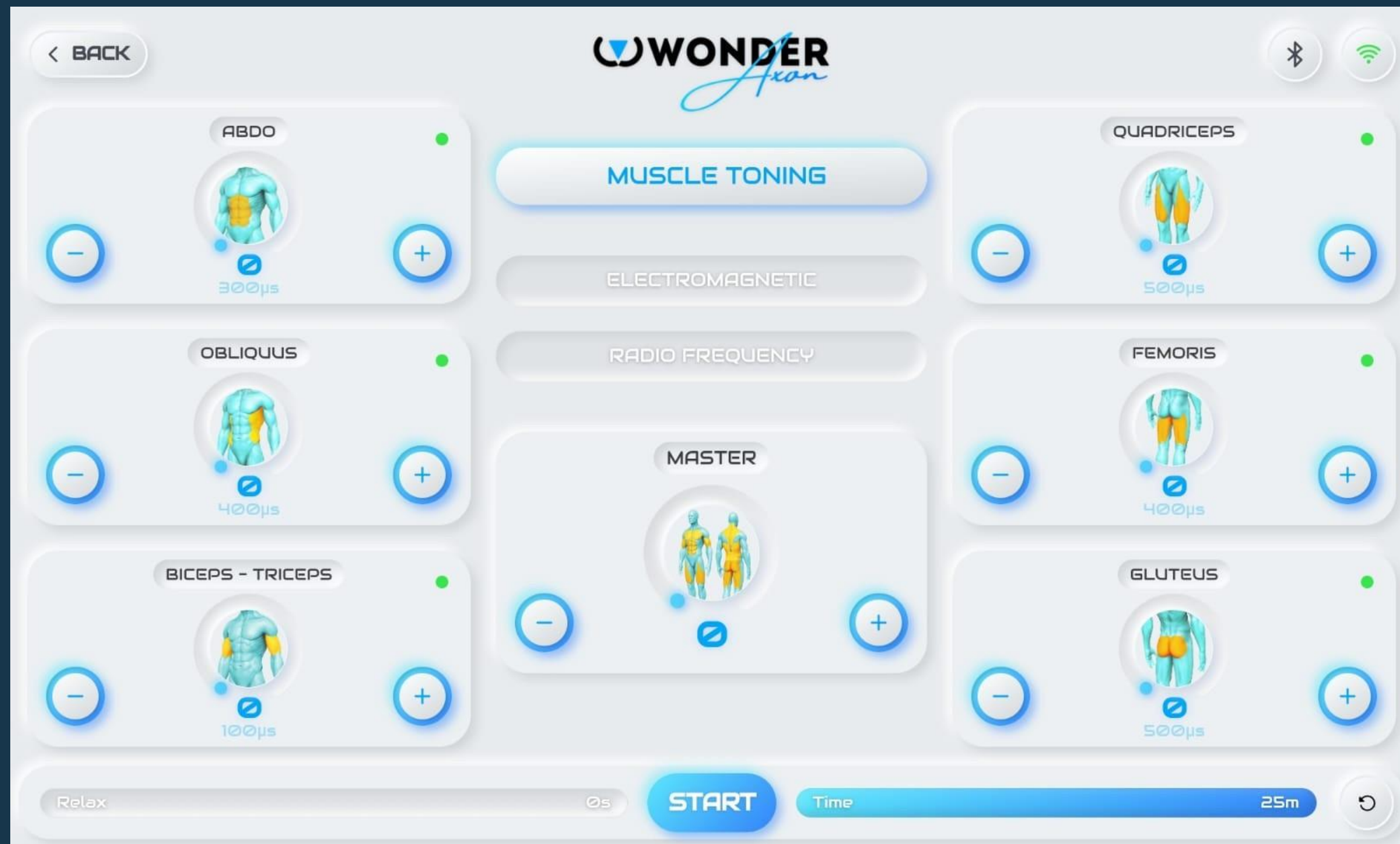
AUTOMATIC PROGRAMS

THEY COMBINE DIFFERENT PARAMETERS THROUGHOUT THE SESSION DEPENDING ON THE OBJECTIVE SOUGHT.

FREQUENCY, PULSE WIDTH AND INTENSITY ARE GRADUALLY INCREASING.

AUTOMATIC PROGRAMS

THEY COMBINE DIFFERENT PARAMETERS THROUGHOUT THE SESSION .
FREQUENCY, PULSE WIDTH AND INTENSITY ARE GRADUALLY INCREASING.

AUTOMATIC PROGRAMS

1 MUSCLE TONING

Muscle development program. Focused on increasing strength and endurance. Create new muscle fibers and improve existing ones.

2 FAT BURNING

Program created to cause a metabolic alteration that triggers a process of burning calories and losing fat.

3 SKIN IMPROVEMENT

Vascular program. Increases blood flow with a muscle pumping effect. Improves cellulite processes and the most recent stretch marks. It produces an effect of tightness and homogeneity of the skin. It has an important lymphatic drainage effect.

4 PELVIC FLOOR

This program is based on the progressive penetration of signals into the structure of muscles and connective tissue that constitutes the suspension support for the pelvic and abdominal organs.

5 ATHLETES

Program created for people who are dedicated to intense and constant physical exercise.

6 REHABILITATION

Program for the improvement of muscular and ligament complications. It has an analgesic effect and has been created for use by physiotherapists. Recommended for the elderly, weak, with muscle atrophies and in need of recovery.

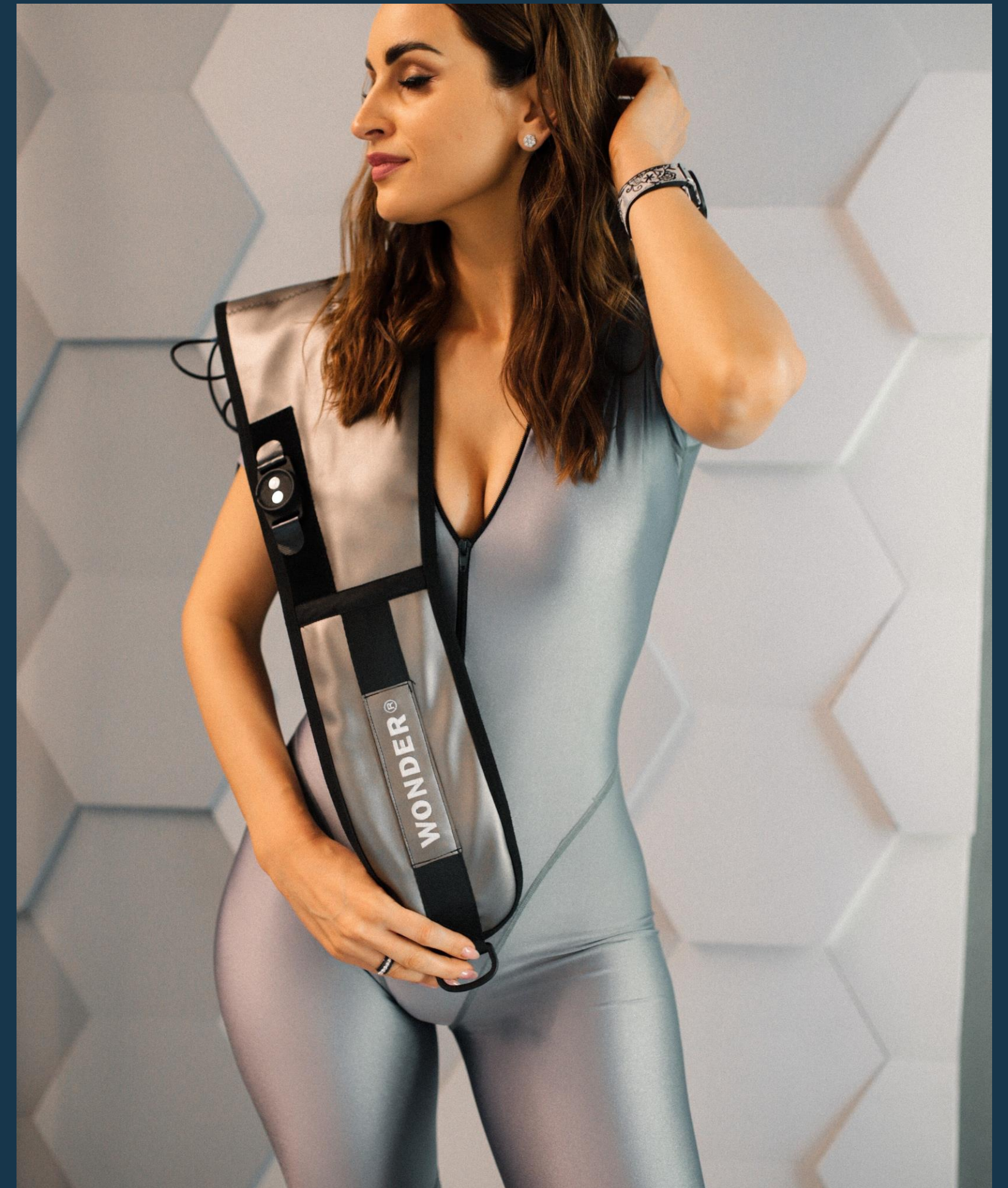
60.000

MUSCLE

CONTRACTIONS

BEST RESULTS

FAST EFFECT

RADIOFREQUENCY IN WONDER AXON:

Monopolar
Inductive
500Khz

Deep action.
40° temperature
15W power

MULTIUSER
TECHNOLOGY

1 DEVICE = 4 USERS

INCREASE
RENTABILITY

6 MUSCLE GROUPS SIMULTANEOUSLY

Legs
Toned

Action
On Obliques

Arms
Strengthened

Quadriceps
Reinforced

Glutes
Lifted

Abdomen
Defined

MEDIAS

iHOLA!

THE COSMETIC SURGERY
MAGAZINE

Semana

zeleb*

HOY
MAGAZINE

mujerhoy

 SPORTRHD.NEWS

KOOSS

ABC

LA GACETA
desalamanca.es

LOOK
MAGAZINE

exclusiva_{ss}

GLAMOUR

BEST! BEAUTY
& CARE #22

Wonder has a presence in the leading media all over the world.

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